

Thank you.

My goal is to make the best mobility products at the best price. As a wheelchair user myself, I know the frustration of ancient technology that costs a fortune, with few real solutions out there.

So I started this nonprofit to change that. The money from this product goes straight to helping people in need and to keep innovating.

It's not about the money. It's about you.

A handwritten signature in black ink, appearing to read 'Ryleigh Newman', with a stylized, cursive script.

Ryleigh Newman

Founder, Rockstar Health

Turn over for setup

**ROCKSTAR
HEALTH.ORG**

Get started.

1 Take off the old one

Gently grab it and twist clockwise and counterclockwise until it comes free. The twisting motion is what matters most.

2 Put on your topper

Line it up with the metal joystick shaft, then twist clockwise and counterclockwise while gently pushing down.

3 Give it a few days

Your hand knows your old joystick by feel, so a new shape takes some adjusting. It took me three days to get used to mine.

Cleaning

A quick wipe with a damp cloth restores its finish.

Questions?

I'm here for you. Email me anytime at Ryleigh@RockstarHealth.org

**ROCKSTAR
HEALTH.ORG**